

Test Result削除方法

実際の運用方法と異なる場合がございます。
この資料は補助的なものとして、詳細は取扱説明書をご参照ください。


はじめに

管理者権限を持っているユーザーに限り、
Test resultを削除することができます。

* データベースには結果は残っているため削除しても復元は可能です。

The screenshot shows the AQUA system interface for a user with administrator privileges. The 'Review' page is active, and the 'Test Results' tab is selected. The search filters are set to 'CHECKMATE' for the test, 'CHECKMATE - VERSAHD' for the test instance, and a date range from 04/01/2020 to 04/30/2020. The 'Test Run List' table displays 12 rows of test results. Rows 170, 171, 172, and 174 are highlighted with a red border, and a large green 'X' is overlaid on the table, indicating that these results have been deleted.

Run ID	Date	Complete	Status	Tester	Revision Of Run
168	04/28/2020	04/28/2020 14:20	FAIL	aquaPhysicist	
170	04/27/2020	04/27/2020 12:50	FAIL	aquaPhysicist	
171	04/27/2020	04/27/2020 12:50	FAIL	aquaPhysicist	
172	04/27/2020	04/27/2020 12:50	FAIL	aquaPhysicist	
174	04/27/2020	04/27/2020 12:50	FAIL	aquaPhysicist	
178	04/27/2020	04/27/2020 17:08	PASS	aquaPhysicist	
179	04/27/2020	04/27/2020 17:26	PASS	aquaPhysicist	
180	04/27/2020	04/27/2020 17:27	FAIL	aquaPhysicist	
181	04/27/2020	04/27/2020 17:28	FAIL	aquaPhysicist	
182	04/27/2020	04/27/2020 17:28	PASS	aquaPhysicist	
183	04/27/2020	04/27/2020 17:38	PASS	aquaPhysicist	
184	04/27/2020	04/27/2020 17:38	FAIL	aquaPhysicist	

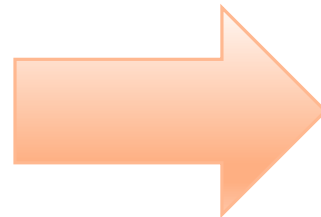

The screenshot shows the same AQUA system interface, but now the 'Test Run List' table only displays 9 rows, indicating that the results from rows 170, 171, 172, and 174 have been successfully deleted. The search filters and other UI elements remain the same.

Run ID	Date	Complete	Status	Tester	Revision Of Run
168	04/28/2020	04/28/2020 14:20	FAIL	aquaPhysicist	
174	04/27/2020	04/27/2020 12:50	FAIL	aquaPhysicist	
178	04/27/2020	04/27/2020 17:08	PASS	aquaPhysicist	
179	04/27/2020	04/27/2020 17:26	PASS	aquaPhysicist	
180	04/27/2020	04/27/2020 17:27	FAIL	aquaPhysicist	
181	04/27/2020	04/27/2020 17:28	FAIL	aquaPhysicist	
182	04/27/2020	04/27/2020 17:28	PASS	aquaPhysicist	
183	04/27/2020	04/27/2020 17:38	PASS	aquaPhysicist	
184	04/27/2020	04/27/2020 17:38	FAIL	aquaPhysicist	

Test Result削除方法

- ① 管理者権限を持つユーザーでログインし、
「System Data」タブ > 「Test Runs」を開きTest項目を選択します。

The screenshot displays the AQUA interface for Tokyo Metropolitan University. At the top, a navigation bar includes a 'System Data' tab (highlighted with a red box), a 'To-Do List' icon, a 'Dashboard' icon, a 'Status' icon, a 'Library' icon, an 'Events' icon, and a 'Review' icon. Below this, the 'System Data' section is active, with a sub-menu containing 'Time Periods', 'Holidays', 'Units', 'Test Runs' (highlighted with a red box), 'Notification', 'License', and 'System Settings'. Underneath, the 'Search Test Runs' section features a search form with a 'Machine' dropdown set to 'VER8AHD', a 'Test' dropdown set to 'CHECKMATE', and a 'Date Range' from '04/01/2020' to '04/28/2020'. A blue 'SEARCH' button is positioned below the form.

②削除したい項目のチェックを外し「SAVE」をクリックします。

Search Test Runs


Machine: VERBAHD Test: CHECKMATE Date Range: 04/01/2020 - 04/28/2020

SEARCH

Resulting Runs

- true 170 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 171 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 172 FAIL 04/27/2020 CHECKMATE - VERBAHD
- false 173 CANCEL 04/27/2020 CHECKMATE - VERBAHD
- true 174 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 175 PASS 04/27/2020 CHECKMATE - VERBAHD
- true 179 PASS 04/27/2020 CHECKMATE - VERBAHD
- true 180 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 181 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 182 PASS 04/27/2020 CHECKMATE - VERBAHD
- true 183 PASS 04/27/2020 CHECKMATE - VERBAHD
- true 184 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 188 FAIL 04/28/2020 CHECKMATE - VERBAHD

SAVE

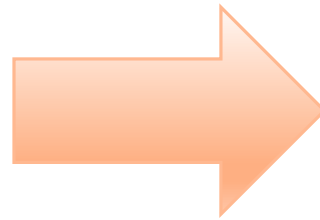

Search Test Runs

Machine: VERBAHD Test: CHECKMATE Date Range: 04/01/2020 - 04/28/2020

SEARCH

Resulting Runs

- true 170 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 171 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 172 FAIL 04/27/2020 CHECKMATE - VERBAHD
- false 173 CANCEL 04/27/2020 CHECKMATE - VERBAHD
- true 174 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 178 PASS 04/27/2020 CHECKMATE - VERBAHD
- true 179 PASS 04/27/2020 CHECKMATE - VERBAHD
- true 180 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 181 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 182 PASS 04/27/2020 CHECKMATE - VERBAHD
- true 183 PASS 04/27/2020 CHECKMATE - VERBAHD
- true 184 FAIL 04/27/2020 CHECKMATE - VERBAHD
- true 188 FAIL 04/28/2020 CHECKMATE - VERBAHD

SAVE

③ 「Review」 > 「Test Result」を開き、テスト結果が削除されているか確認します。

The screenshot shows the AQUA interface for TOKYO METROPOLITAN UNIVERSITY. The top navigation bar includes icons for To-Do List, Dashboard, Status, Library, Events, and Review. The 'Review' button is highlighted with a red box. Below the navigation bar, the 'Review' section is active, with 'Test Results' highlighted in a red box. The search filter section includes a dropdown for 'Test' (CHECKMATE), a checkbox for 'Include Draft Tests', a dropdown for 'Test Instance' (CHECKMATE - VERBAHD), a 'Date Range' (04/01/2020 - 04/30/2020), and radio buttons for 'Test Runs' (Non-Draft selected, Draft). A 'SEARCH' button and 'No Plot Available' message are also present. Below the search filter is the 'Test Run List' table.

Run ID	Date	Complete	Status	Tester	Revision Of Run
188	04/28/2020	04/28/2020 14:20	FAIL	equePhysicist	
174	04/27/2020	04/27/2020 12:50	FAIL	equePhysicist	
178	04/27/2020	04/27/2020 17:08	PASS	equePhysicist	
179	04/27/2020	04/27/2020 17:26	PASS	equePhysicist	
180	04/27/2020	04/27/2020 17:27	FAIL	equePhysicist	
181	04/27/2020	04/27/2020 17:28	FAIL	equePhysicist	
182	04/27/2020	04/27/2020 17:28	PASS	equePhysicist	
183	04/27/2020	04/27/2020 17:38	PASS	equePhysicist	
184	04/27/2020	04/27/2020 17:38	FAIL	equePhysicist	

Showing 1 - 9 of 9

Thank you
