AQUA

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Topic : 1

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The screenshot displays the 'AQUA' software interface in 'Test List > Edit Test' mode. The user is logged in as 'aquaphysicist'. The navigation menu includes 'Tests', 'Scheduling', 'Equipment', 'Compliance', and 'System Data'. The main toolbar contains icons for 'To-Do List', 'Dashboard', 'Status', 'Library', 'Events', and 'Review'. The 'Tests' section has buttons for 'Test List', 'Test Groups', 'File Upload', and 'Test Merge'. The current test being edited has the following details:

- ID:** 2356
- Name:** Output Constancy
- Version:** 6
- Provenance:** Acumyn
- Status:** Draft

The form includes expandable sections for 'Description', 'Script', 'Procedure', 'Specification', and 'Test Images'. Below these are fields for 'Parent Tests' (Output and Backup Monitor Chamber Constancy), 'Child Tests' (-), and 'Test Groups' (Compliance Report Weekly and Monthly QC Tests). The 'Script Type' is set to 'XML', and there is a 'Show Parameter' checkbox. At the bottom, there are buttons for 'SAVE', 'EXPORT', 'PREVIOUS VERSION', and 'CANCEL'.

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The screenshot displays the 'Scheduling' section of the AQUA software interface. At the top, there is a navigation bar with the user 'aquaphysicist' and tabs for 'Tests', 'Scheduling', 'Equipment', 'Compliance', and 'System Data'. The 'Scheduling' tab is active. Below the navigation bar, there are icons for 'To-Do List', 'Dashboard', 'Status', 'Library', 'Events', and 'Review'. The main content area is titled 'Scheduling' and features a search bar. Below the search bar, there are four dropdown menus for filtering: 'Test' (set to '(None)'), 'Equipment' (set to 'VersaHD A'), 'User or Group' (set to '(None)'), and 'Time Period' (set to '(None)'). A blue 'SEARCH' button is positioned below these filters, and a 'NEW SCHEDULE' button is located below it. The 'Schedule Search Results' section contains a table with the following data:

User or Group	Period	Test Instance	List Order
Acumyn Linac Therapist	Daily	Therapist Daily All Linacs - VersaHD A	1000
Acumyn Linac Therapist	Daily	Therapists Daily Tests - Elekta All - VersaHD A	1000
Acumyn Linac Therapist	Daily	KV-MV Iso Check - VersaHD A	1000

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The screenshot displays the AQUA Compliance management interface. The navigation bar includes 'aquaphysicist', 'Tests', 'Scheduling', 'Equipment', 'Compliance' (selected), and 'System Data'. The main header shows 'AQUA' and 'AQUA TRAINING'. The 'Compliance' section has two tabs: 'Machine Compliance' (selected) and 'Policy Compliance'. Under 'Machine Compliance', there is a 'Machine' dropdown menu currently set to 'VersaHD A'. Below this is a 'Compliance Elements' table with the following data:

Section Name	Section Type	Test Group	Order
Mandatory Safety	Mandatory Safety	Mandatory Safety CNSC Non-TrueBeam	0
Daily Safety Tests	Report Section	Compliance Report Daily Safety Tests	1
Daily QC Tests	Report Section	Compliance Report Daily QC Tests	2
Weekly and Monthly QC Tests	Report Section	Compliance Report Weekly and Monthly QC Tests	3

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The screenshot shows the 'System Data' section of the AQUA TRAINING application. The user 'aquaphysicist' is logged in. The navigation menu includes Tests, Scheduling, Equipment, Compliance, and System Data (which is active). The main content area has tabs for Time Periods, Holidays, Units, Test Runs, Notification, License, and System Settings. The 'Time Periods' tab is selected, displaying a table with the following data:

Name	Warning Days	Late Days	Default Plot Days	Fully Compliant (%)	Acceptable (%)	Requires Follow-Up (%)
Daily	0	1	30	≥ 100.0	-	< 100.0
Weekly	1	1	60	≥ 100.0	-	< 100.0
Every 2 Weeks	4	1	120	≥ 100.0	-	< 100.0

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The screenshot displays the 'Edit Group' interface for 'AQUA Admin'. The 'Name' field is highlighted with a red box. Below the main form are 'SAVE', 'DELETE', and 'CANCEL' buttons. The 'Parents' table shows 'AQUA User' as the parent. The 'Children' table lists 'AQUA Debug', 'aquaPhysicist', and 'Test Hospital Admin', with the latter highlighted in red. A modal window titled 'SELECTION' is open, showing a list of 'Available Items' and 'Selected Items'. 'Test Hospital Admin' is selected in the 'Selected Items' list and highlighted with a red box. The modal also has 'OK' and 'CANCEL' buttons.

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The screenshot shows the 'Edit Group' form for 'Test Hospital Admin'. The form fields are:

- Type: GROUP
- Id: 306
- Name: Test Hospital Admin
- Description: (empty)

Buttons: SAVE, DELETE, CANCEL

Parents

Id	Name	Type
6	ADMIN	GROUP
9	AQUA User	GROUP
10	AQUA Admin	GROUP
11	AQUA Test Writers	GROUP
13	AQUA Reporting	GROUP
304	AQUA Machine Status	GROUP

The screenshot shows the 'Edit Group' form for 'Test Hospital User'. The form fields are:

- Type: GROUP
- Id: 307
- Name: Test Hospital User
- Description: (empty)

Buttons: SAVE, DELETE, CANCEL

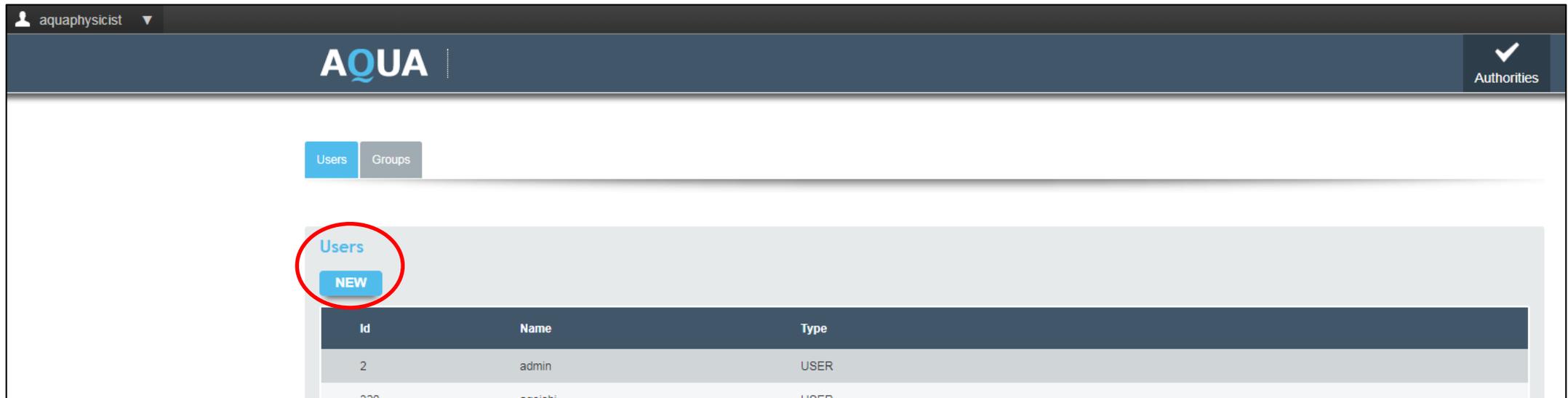
Parents

Id	Name	Type
9	AQUA User	GROUP
11	AQUA Test Writers	GROUP
13	AQUA Reporting	GROUP
304	AQUA Machine Status	GROUP

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The screenshot shows the AQUA user management interface. At the top, there is a user profile dropdown for 'aquaphysicist' and the AQUA logo. Below the logo, there are two tabs: 'Users' (active) and 'Groups'. Under the 'Users' tab, there is a 'NEW' button circled in red. Below the button is a table with columns 'Id', 'Name', and 'Type'. The table contains two rows: one with '2' for Id, 'admin' for Name, and 'USER' for Type; and another with '220' for Id, 'aquaphysicist' for Name, and 'USER' for Type.

Id	Name	Type
2	admin	USER
220	aquaphysicist	USER

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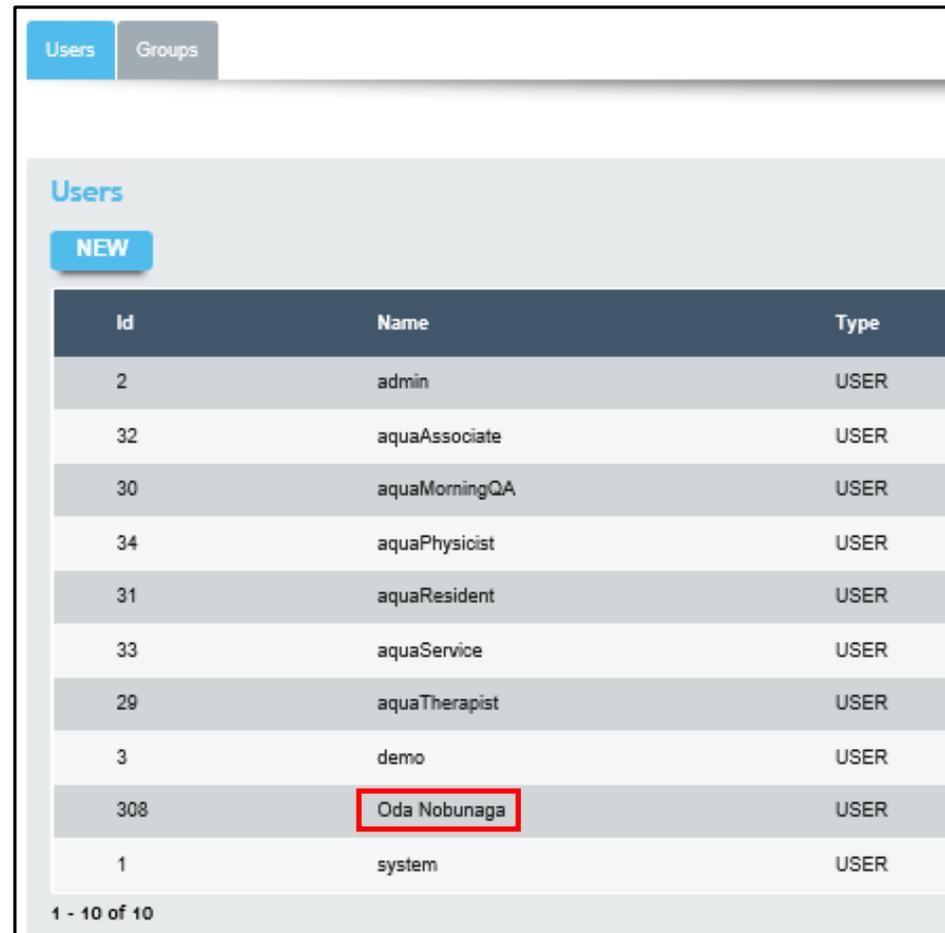
The screenshot displays the 'Edit User' form in the Elekta application. The form is organized into several sections:

- Navigation:** 'Users' and 'Groups' tabs are visible at the top left.
- Breadcrumbs:** 'Users > Edit User' is shown below the navigation tabs.
- Form Fields:**
 - Type:** A dropdown menu set to 'USER'.
 - Id:** A text input field containing the value '308'.
 - Account Name:** A text input field containing 'Oda Nobunaga'.
 - First:** A text input field containing 'Nobunaga'.
 - Middle:** An empty text input field.
 - Last:** A text input field containing 'Oda'.
 - Account Type:** Radio buttons for 'Local' (selected) and 'Remote'.
 - PASSWORD:** A password input field with masked characters '****'.
 - Confirm PASSWORD:** A password input field with masked characters '****' and a confirmation icon.
 - Email Address:** A text input field containing 'User@elekta.com'.
 - Parents:** A text input field.
- Buttons:** A large blue 'EDIT' button is centered below the form fields. At the bottom left, there are three buttons: 'SAVE' (blue), 'DELETE' (red), and 'CANCEL' (grey).

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The screenshot shows a user management interface with a 'Users' tab selected. A 'NEW' button is visible. Below it is a table listing users with columns for Id, Name, and Type. The user 'Oda Nobunaga' (Id: 308) is highlighted with a red box.

Id	Name	Type
2	admin	USER
32	aquaAssociate	USER
30	aquaMorningQA	USER
34	aquaPhysicist	USER
31	aquaResident	USER
33	aquaService	USER
29	aquaTherapist	USER
3	demo	USER
308	Oda Nobunaga	USER
1	system	USER

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The screenshot shows a user management interface. On the left, there are two tables: 'Parents' and 'Children'. The 'Children' table is highlighted with a red box and contains one entry: Oda Nobunaga (USER). In the center, a 'SELECTION' dialog box is open, showing a list of 'Available Items' and a 'Selected Items' list. 'Oda Nobunaga' is selected in the 'Selected Items' list and highlighted with a red box. The 'EDIT' button is visible at the top right of the dialog box. At the bottom of the dialog box, there are 'OK' and 'CANCEL' buttons.

Id	Name	Type
6	ADMIN	GROUP
9	AQUA User	GROUP
10	AQUA Admin	GROUP
11	AQUA Test Writers	GROUP
13	AQUA Reporting	GROUP
304	AQUA Machine Status	GROUP

Id	Name	Type
308	Oda Nobunaga	USER

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The screenshot displays the 'Edit User' interface. At the top, there are tabs for 'Users' and 'Groups'. Below the breadcrumb 'Users > Edit User', the form is organized into three columns:

- Left Column:** 'Type' (USER), 'First' (Nobunaga), 'Account Type' (Local selected, Remote unselected), and 'Email Address' (User@elekta.com).
- Middle Column:** 'Id' (308), 'Middle' (empty), 'PASSWORD' (empty), and 'Parents' (Test Hospital Admin, highlighted with a red box).
- Right Column:** 'Account Name' (Oda Nobunaga), 'Last' (Oda), and 'Confirm PASSWORD' (empty).

At the bottom of the form, there are four buttons: 'SAVE' (blue), 'DELETE' (red), 'CANCEL' (dark grey), and 'EDIT' (blue).

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Equipment

Machines Machine Types Machine Groups Machine Components Machine Reference Workstations Interface

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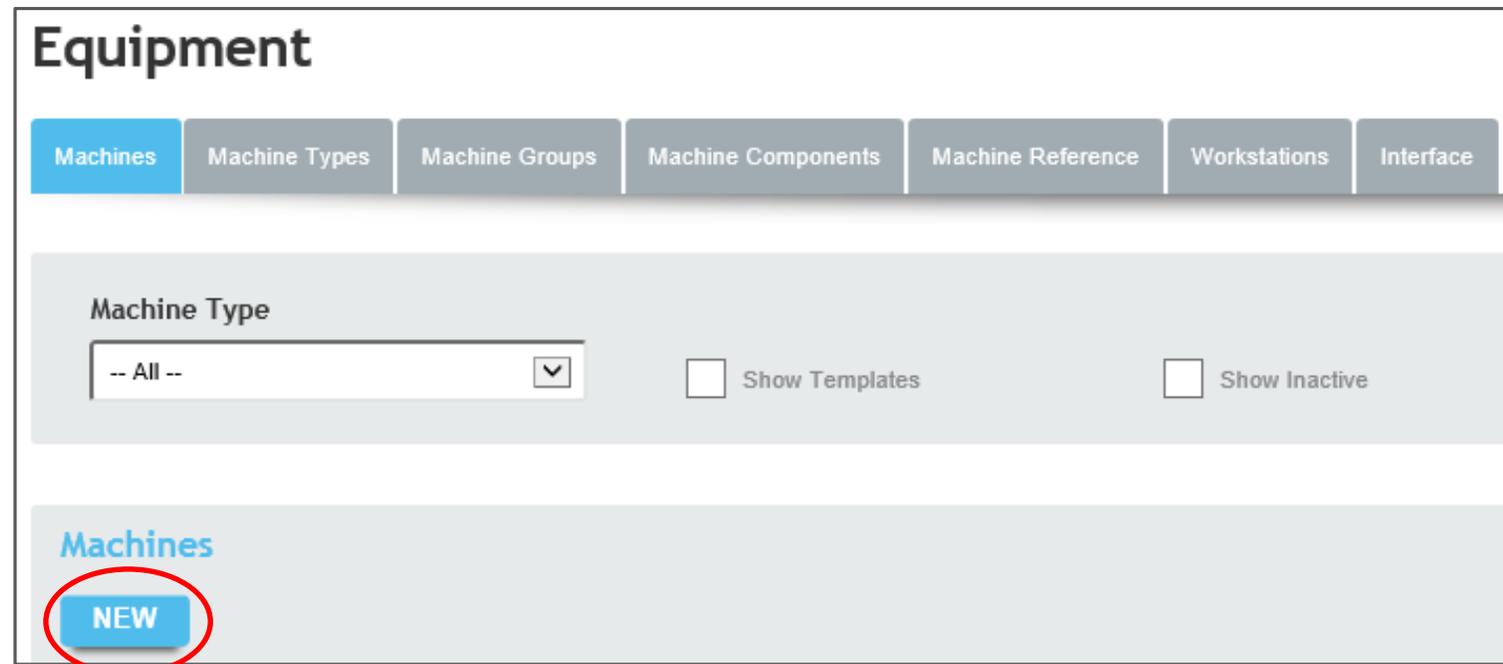
Serial Number: 150001

Machine as Template: -- Select --

Machine Components Machine Components & Test Instances Machine Components, Test Instances & Schedules

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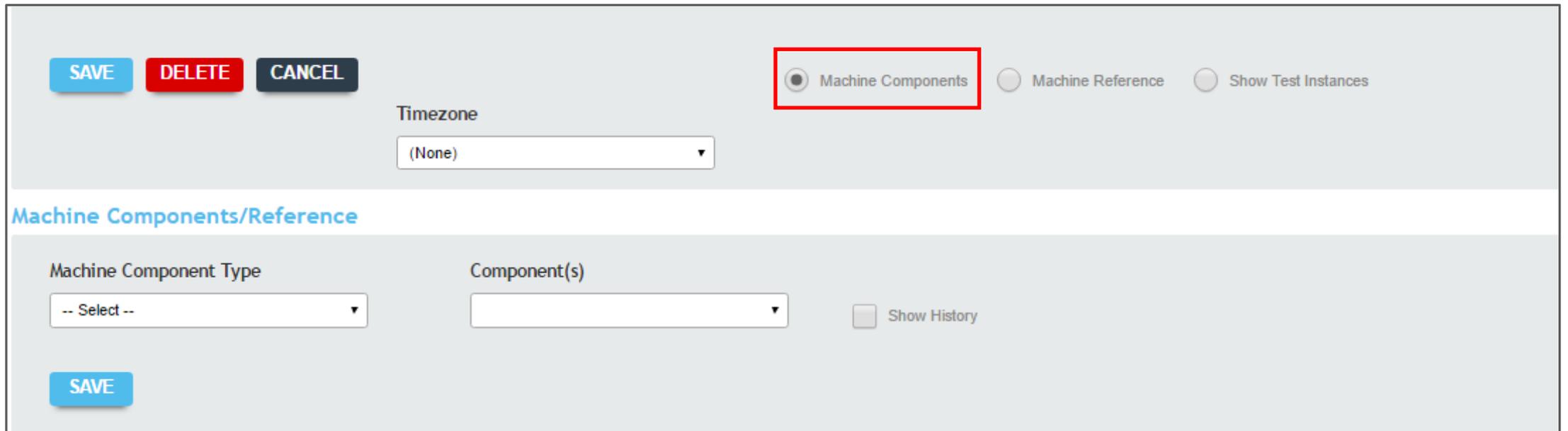
The screenshot displays a web interface for defining a device. At the top, there are three buttons: 'SAVE' (blue), 'DELETE' (red), and 'CANCEL' (dark grey). To the right, there are three radio buttons: 'Machine Components' (selected and highlighted with a red box), 'Machine Reference', and 'Show Test Instances'. Below these is a 'Timezone' dropdown menu currently set to '(None)'. A section header 'Machine Components/Reference' is visible. Underneath, there are two dropdown menus: 'Machine Component Type' (set to '-- Select --') and 'Component(s)'. To the right of these is a 'Show History' checkbox. A 'SAVE' button is located at the bottom left of the form area.

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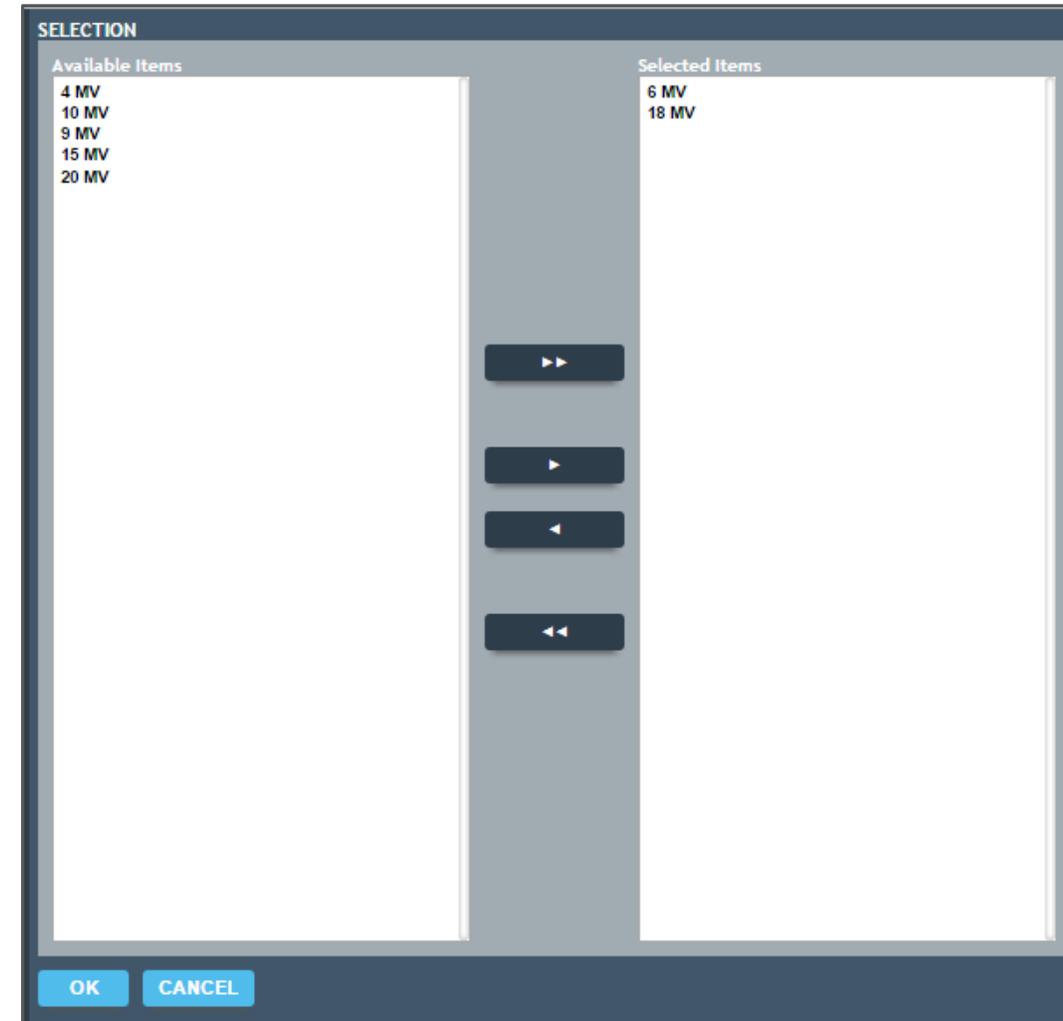
Machine Component Type	登録例	Description
Delivery Photon Energy	6 MV, 10 MV	Available photon energies on the linac
Delivery Electron Energy	6 MeV, 9 MeV	Available electron energies on the linac
Delivery FFF Photon Energy	6 FFF MV, 10 FFF MV	Available flattening filter free energies on the linac
Imaging Photon Energy	70 kVp, 110 kVp	Available imaging photon energies on the linac
TBI Mode	TBI	Map this component if the linac is used for TBI
TSET Mode	TSET	Map this component if the linac is used for TSET
Wedge	Universal, Dynamic, Physical	Map this component if the wedges are used on the linac
Wedge Angle	15 Deg, 30 Deg, 45 Deg	Map this component if the wedges are used on the linac
Electron Applicators	6x6 cm ² , 6x10 cm ² , 10x10 cm ²	Map the available Electron Applicator sizes

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Machine Component Type	Setting
Linear Accelerator Model	VersaHD
Mode	IMRT
Delivery Photon Energy	6 MV
Delivery Photon Energy	10 MV
Delivery Photon Energy	4 MV
Delivery Electron Energy	9 MeV
Delivery Electron Energy	12 MeV
Delivery Electron Energy	6 MeV
Delivery Electron Energy	4 MeV
Delivery Electron Energy	15 MeV
Delivery FFF Photon Energy	6 MV FFF
Delivery FFF Photon Energy	10 MV FFF
Imaging Photon Energy	70 kVp
Imaging Photon Energy	110 kVp
Vendor	Elekta

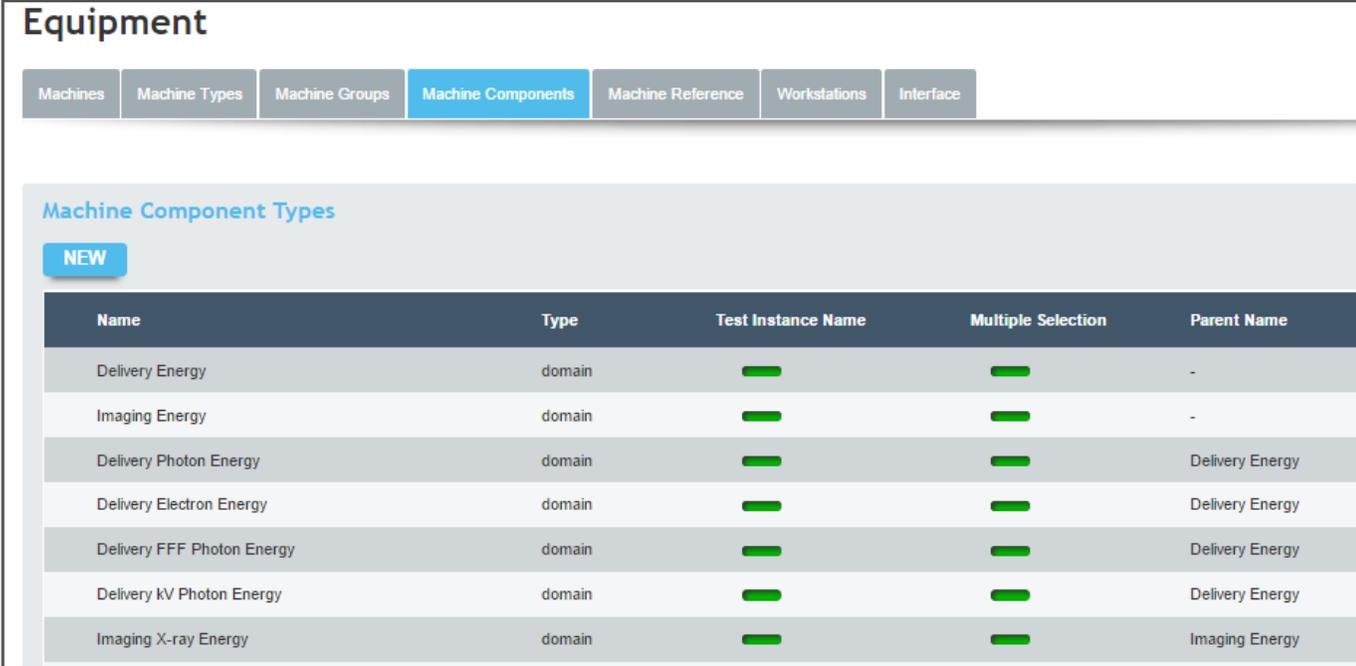
Model	Versa HD
TBI Mode	TBI
Wedge	Universal
Wedge Angle	60 Deg
Collimator	Agility
Megavoltage flat-panel detector	Elekta aSi
Kilovoltage Imaging	XVI 5.0
Electron Applicators	6x6 cm2
Electron Applicators	10x10 cm2
Electron Applicators	14x14 cm2
Electron Applicators	20x20 cm2
Electron Applicators	25x25 cm2
Treatment Couch	HexaPOD Couch
Gating System	Catalyst
Special Treatment Types	TBI

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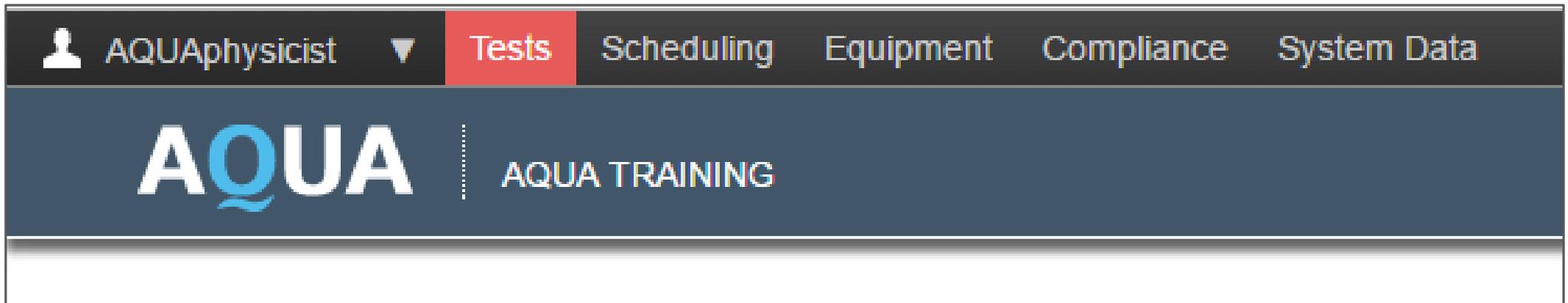
The screenshot shows the 'Equipment' management interface. At the top, there are several tabs: 'Machines', 'Machine Types', 'Machine Groups', 'Machine Components' (which is selected and highlighted in blue), 'Machine Reference', 'Workstations', and 'Interface'. Below the tabs, there is a section titled 'Machine Component Types' with a 'NEW' button. A table lists various component types with columns for Name, Type, Test Instance Name, Multiple Selection, and Parent Name. Each row has a green bar in the Test Instance Name and Multiple Selection columns.

Name	Type	Test Instance Name	Multiple Selection	Parent Name
Delivery Energy	domain	█	█	-
Imaging Energy	domain	█	█	-
Delivery Photon Energy	domain	█	█	Delivery Energy
Delivery Electron Energy	domain	█	█	Delivery Energy
Delivery FFF Photon Energy	domain	█	█	Delivery Energy
Delivery kV Photon Energy	domain	█	█	Delivery Energy
Imaging X-ray Energy	domain	█	█	Imaging Energy

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The screenshot shows a web interface for test configuration. At the top, there is a search bar labeled 'Test Name' containing the text 'flatness'. To the right of the search bar is a checkbox labeled 'Include Draft' which is currently unchecked. Below the search bar is a blue button labeled 'SEARCH'. Below the 'SEARCH' button is a black button labeled 'NEW BINARY TEST'. Below these elements is a section titled 'Test List' which contains a table with the following data:

Name	Version	Final	Test ID
BeamChecker Output Flatness Symmetry	2		2234
DQA3 Lite Output Flatness Symmetry	2		2235
Daily Calibration/Flatness	2		1763
Flatness and Symmetry	3		2268
Flatness and Symmetry ICProfiler	1		2214

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Background: [Dropdown] Foreground: [Dropdown] Font: [Dropdown] Size: [Dropdown]

Compare the measured PDD with the reference data.

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The screenshot shows a configuration panel with the following elements:

- Parent Tests**: -
- Child Tests**: -
- Test Groups**: Compliance Report Weekly and Monthly QC Tests
- Script Type**: XML (dropdown menu)
- Show Parameter

At the bottom, there are five buttons: **SAVE** (blue), **EXPORT** (blue), **PREVIOUS VERSION** (blue), **DELETE** (red), and **CANCEL** (dark blue).

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Script Type: XML Show Parameter

SAVE EXPORT PREVIOUS VERSION DELETE CANCEL

Test Parameters

Label	Type	Name	Description
Delivery Energy	equipment	energy	Energy Level.
Fail Tolerance	test	failTolerance	Failing tolerance. Measurements greater than the failing tolerance are flagged as failures.
Radial Flatness	input	radialFlatness	Radial flatness measurement.
Radial Flatness Baseline	test	radialFlatnessBaseline	Baseline value for radial flatness measurement.
Radial Symmetry	input	radialSymmetry	Radial symmetry measurement.
Radial Symmetry Baseline	test	radialSymmetryBaseline	Baseline value for radial symmetry measurement.
Test Result	output	overallResult	Final test status.

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Machine

NEW TEST INSTANCE

Name	Complete	Scheduled
Flatness and Symmetry - Test Hospital VersaHD (4 MV)		
Flatness and Symmetry - Test Hospital VersaHD (6 MV)		
Flatness and Symmetry - Test Hospital VersaHD (10 MV)		

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Test List > Edit Test > Edit Test Instance

ID	26343	Name	Flatness and Symmetry - Test Hospital VersaHD (4 MV)	Equipment/Machine	Test Hospital VersaHD	Complete	true
----	-------	------	--	-------------------	-----------------------	----------	------

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The screenshot shows a web interface for creating a new binary test. At the top, there is a form with a 'Test Name' input field containing the text 'flatness'. A red circle highlights the input field, and a red arrow points from the text '「flatness」と入力し、絞り込みます' to the input field. To the right of the input field is a checkbox labeled 'Include Draft'. Below the input field is a blue 'SEARCH' button and a black 'NEW BINARY TEST' button. Below the form is a 'Test List' table with the following data:

Name	Version	Final	Test ID
BeamChecker Output Flatness Symmetry	2	■	2234
DQA3 Lite Output Flatness Symmetry	2	■	2235
Daily Calibration/Flatness	2	■	1763
Flatness and Symmetry	3	■	2268

The 'Flatness and Symmetry' row in the table is highlighted with a red box.

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The screenshot displays a web interface for configuring tests. At the top, the 'Test Groups' section is set to 'Compliance Report Weekly and Monthly QC Tests'. Below this, the 'Script Type' is set to 'XML' in a dropdown menu, and there is an unchecked checkbox for 'Show Parameter'. A row of five buttons is visible: 'SAVE' (blue), 'EXPORT' (blue), 'PREVIOUS VERSION' (blue), 'DELETE' (red), and 'CANCEL' (dark blue). The 'Test Instances' section below contains a 'Machine' dropdown menu currently set to '-- All --'. At the bottom left of this section, a blue button labeled 'NEW TEST INSTANCE' is circled in red, indicating the target for the user's action.

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The screenshot shows a web interface for creating a new test instance. At the top, there are navigation tabs: 'Test List' (active), 'Test Groups', 'File Upload', and 'Test Merge'. Below the tabs, the breadcrumb path is 'Test List > Edit Test > New Test Instance'. The main form area contains two dropdown menus. The first, labeled 'Equipment/Machine', has 'Test Hospital VersaHD' selected and is highlighted with a red rectangular box. The second, labeled 'Use Test Instance as Template', has '-- Select --' selected. At the bottom of the form, there are two buttons: 'CREATE' (highlighted with a red oval) and 'BACK'.

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Name	Complete
Flatness and Symmetry - Test Hospital VersaHD (4 MV)	<input checked="" type="checkbox"/>
Flatness and Symmetry - Test Hospital VersaHD (6 MV)	<input checked="" type="checkbox"/>
Flatness and Symmetry - Test Hospital VersaHD (10 MV)	<input checked="" type="checkbox"/>

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Test List > Edit Test > Edit Test Instance

ID	10472	Name	Flatness and Symmetry - Test Hospital VersaHD (4 MV)	Equipment/Machine	Test Hospital VersaHD	Complete	true
----	-------	------	--	-------------------	-----------------------	----------	------

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Test List > Edit Test > Edit Test Instance > New Schedule

Test Instance
Flatness and Symmetry - Test Hospital VersaHD (4 MV)

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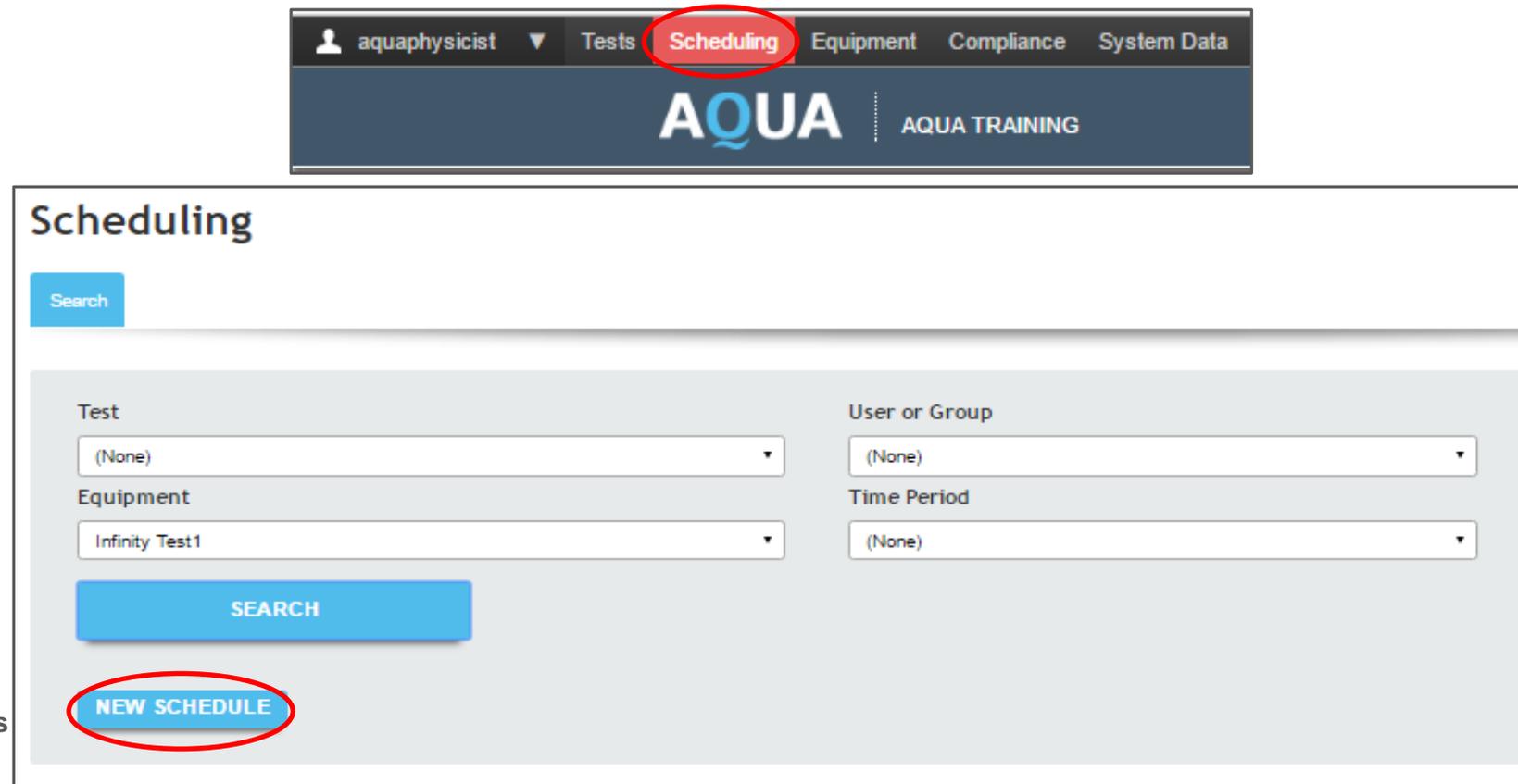
The screenshot displays the AQUA Scheduling interface. At the top, a navigation bar includes a user profile 'aquaphysicist', a dropdown menu, and tabs for 'Tests', 'Scheduling', 'Equipment', 'Compliance', and 'System Data'. The 'Scheduling' tab is highlighted with a red circle. Below the navigation bar, the 'AQUA' logo and 'AQUA TRAINING' text are visible. The main content area is titled 'Scheduling' and contains a search bar. Below the search bar, there are four dropdown menus: 'Test' (set to '(None)'), 'User or Group' (set to '(None)'), 'Equipment' (set to 'Infinity Test1'), and 'Time Period' (set to '(None)'). A blue 'SEARCH' button is positioned below these dropdowns. At the bottom left of the form, a blue button labeled 'NEW SCHEDULE' is circled in red.

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Schedule Search > New Schedule

Test
Flatness and Symmetry

Test Instance
Flatness and Symmetry - Test Hospital VersaHD (6 MV)

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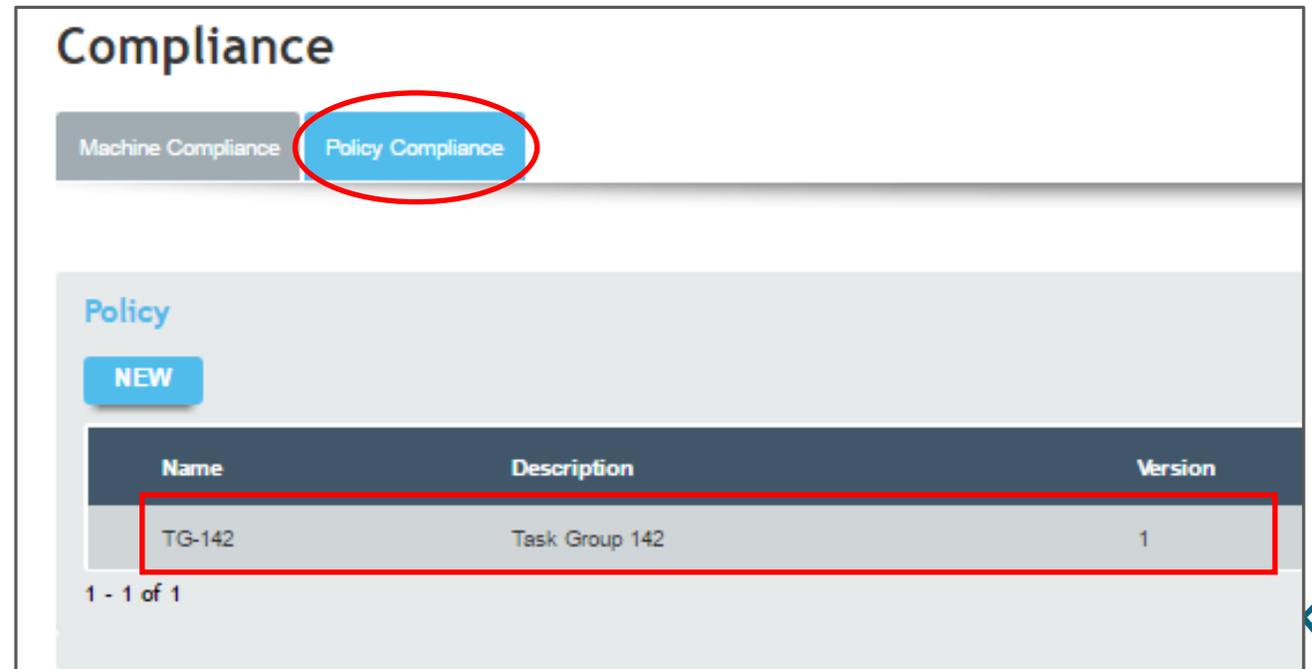
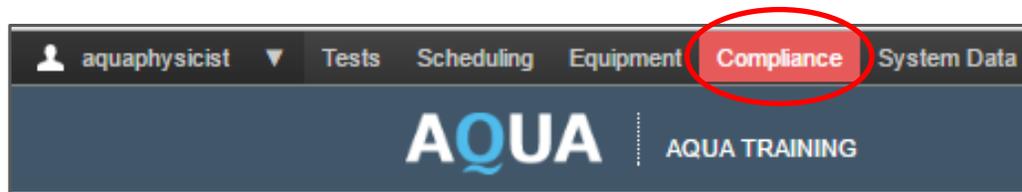
Policy > Map Policy

Name	TG-142
Version	1
Published	09/01/2009
Description	Task Group 142

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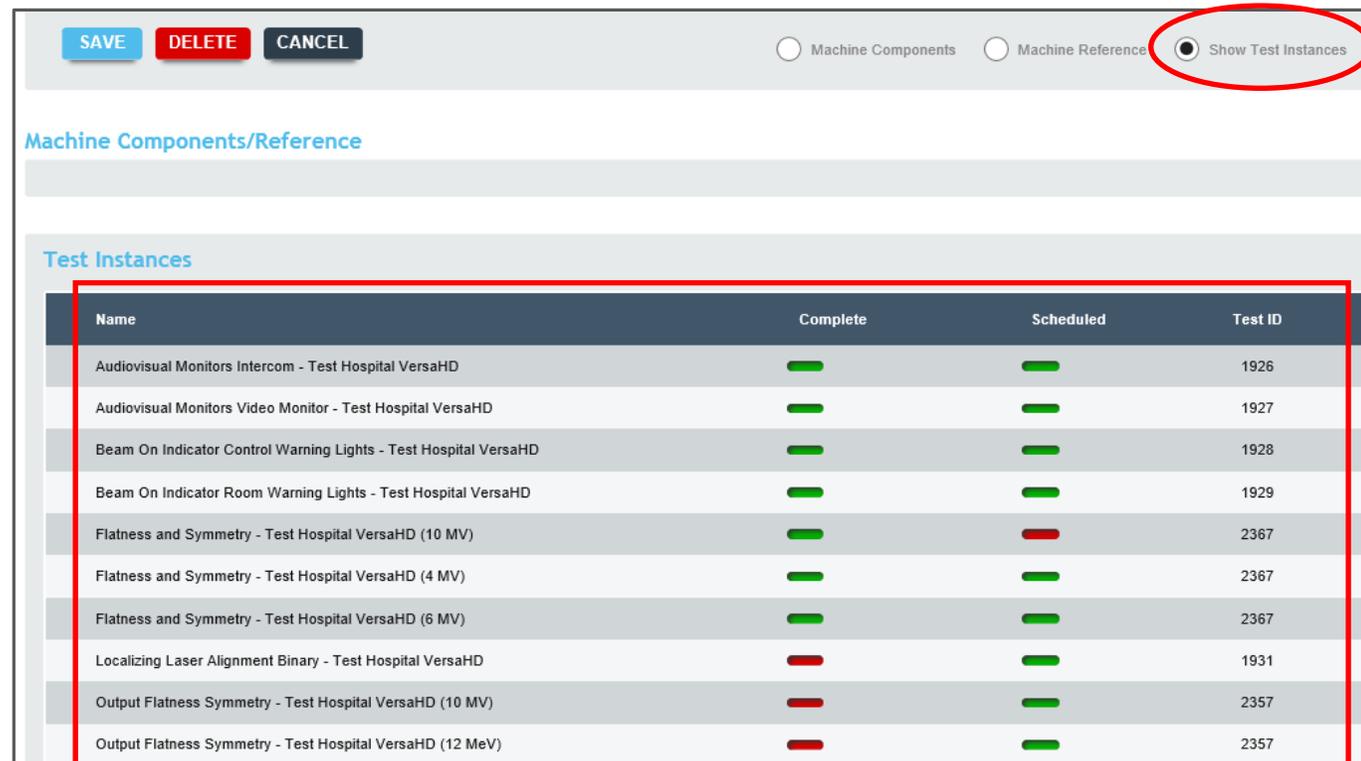
The screenshot shows a web interface for managing test instances. At the top, there are three buttons: 'SAVE' (blue), 'DELETE' (red), and 'CANCEL' (dark grey). To the right, there are three radio buttons: 'Machine Components', 'Machine Reference', and 'Show Test Instances' (which is selected and circled in red). Below the buttons, there is a section titled 'Machine Components/Reference' which is currently empty. Below that, there is a section titled 'Test Instances' which contains a table with the following data:

Name	Complete	Scheduled	Test ID
Audiovisual Monitors Intercom - Test Hospital VersaHD	—	—	1926
Audiovisual Monitors Video Monitor - Test Hospital VersaHD	—	—	1927
Beam On Indicator Control Warning Lights - Test Hospital VersaHD	—	—	1928
Beam On Indicator Room Warning Lights - Test Hospital VersaHD	—	—	1929
Flatness and Symmetry - Test Hospital VersaHD (10 MV)	—	—	2367
Flatness and Symmetry - Test Hospital VersaHD (4 MV)	—	—	2367
Flatness and Symmetry - Test Hospital VersaHD (6 MV)	—	—	2367
Localizing Laser Alignment Binary - Test Hospital VersaHD	—	—	1931
Output Flatness Symmetry - Test Hospital VersaHD (10 MV)	—	—	2357
Output Flatness Symmetry - Test Hospital VersaHD (12 MeV)	—	—	2357

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The screenshot shows the 'Tests' management interface. At the top, there are navigation tabs: 'Test List' (active), 'Test Groups', 'File Upload', and 'Test Merge'. Below the tabs is a search form with a 'Test Name' input field containing 'audio', an 'Include Draft' checkbox, and a blue 'SEARCH' button. A 'NEW BINARY TEST' button is also visible. Below the search form is a 'Test List' table with the following data:

Name	Version	Final	Test ID
Audiovisual Monitors Intercom	8	Final (green bar)	1926
Audiovisual Monitors Video Monitor	8	Final (green bar)	1927

At the bottom of the table, it says 'Showing 1 - 2 of 2'. The 'Final' column header and the green bars in the rows are highlighted with red boxes in the original image.

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AQUA DEMO SYSTEM - DO NOT USE IN A CLINICAL ENVIRONMENT

✓ To-Do List Dashboard Status Library Events Review

To-Do List

Machine Type: -- All -- Machine: Test Hospital VersaHD

Linear Accelerator Test Hospital VersaHD (Elekta VersaHD) Machine Off

Machine Compliance (CNSC Mandatory Tests)

MANDATORY TESTS INCOMPLETE

Mandatory Tests

Test Name	Completed
Audiovisual Monitors Intercom	—
Audiovisual Monitors Video Monitor	—

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The screenshot displays the AQUA dashboard interface. At the top, there is a navigation bar with the AQUA logo, a warning "DEMO SYSTEM - DO NOT USE IN A CLINICAL ENVIRONMENT", and several menu items: To-Do List, Dashboard (highlighted in red), Status, Library, Events, and Review. Below the navigation bar, the main content area is titled "Dashboard". It features a "Machine Group" dropdown menu set to "Linac" and an "As-Of Date" input field. There is also a checkbox labeled "Enable Refresh" which is checked. Below these controls is a table with the following columns: Machine, Effective Date, Machine Status, Mandatory, Compliance, and Performance. The table contains one row of data for "Test Hospital VersaHD" with an effective date of "03/10/2020" and a status of "Machine Off". The "Mandatory" and "Compliance" columns for this row are highlighted with a red box, showing progress indicators. The "Performance" column shows a green bar.

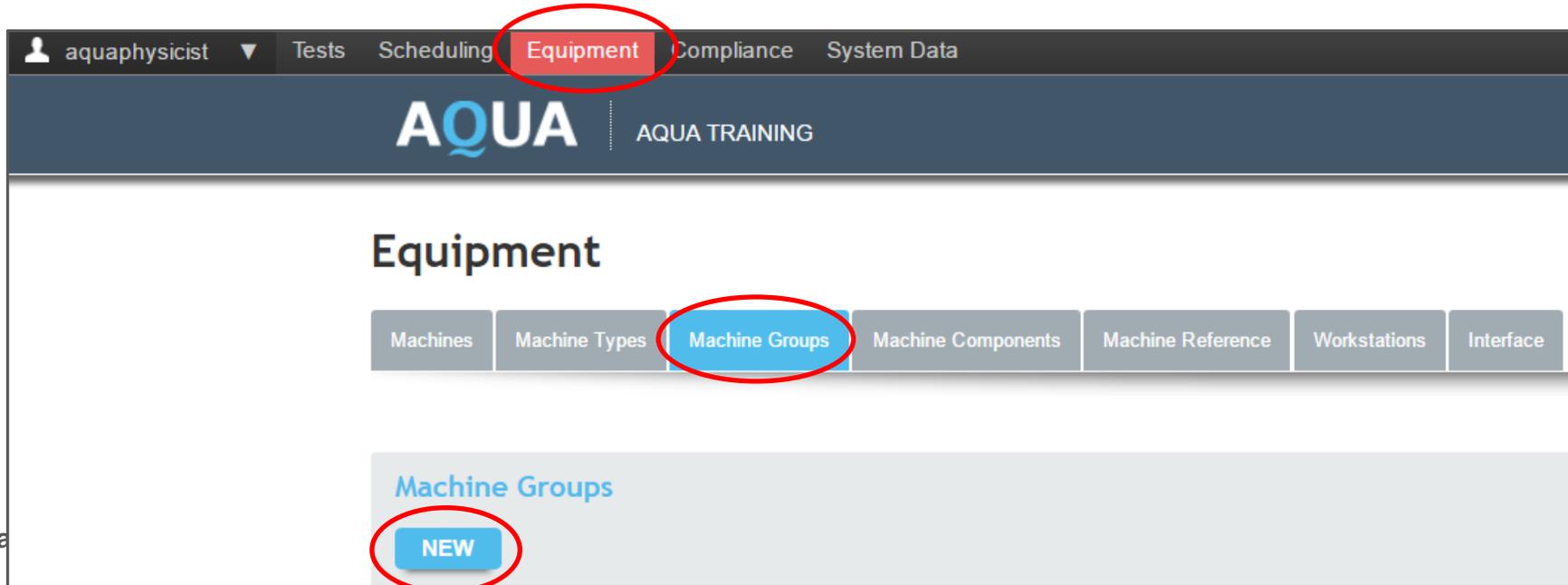
Machine	Effective Date	Machine Status	Mandatory	Compliance	Performance
Test Hospital VersaHD	03/10/2020	Machine Off	<input type="checkbox"/>	<input type="checkbox"/>	<div style="width: 100%; height: 10px; background-color: green;"></div>

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Machines Machine Types **Machine Groups** Machine Components Machine Reference Workstations Interface

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The screenshot displays the AQUA dashboard interface. At the top, the header includes the AQUA logo, a warning "DEMO SYSTEM - DO NOT USE IN A CLINICAL ENVIRONMENT", and navigation tabs for To-Do List, Dashboard (highlighted), Status, Library, Events, and Review. Below the header, the "Dashboard" section contains a "Machine Group" dropdown menu with "Linac" selected, an "As-Of Date" input field, and a checked "Enable Refresh" checkbox. A table below lists machine details:

Machine	Effective Date	Machine Status	Mandatory	Compliance	Performance
Test Hospital VersaHD	03/10/2020	Machine Off			

Thank you