Unity Physics Training

Introduction to Monaco

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Lesson Objectives

Topics Covered

- File sharing between offline and online Monaco
- Monaco file structure

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
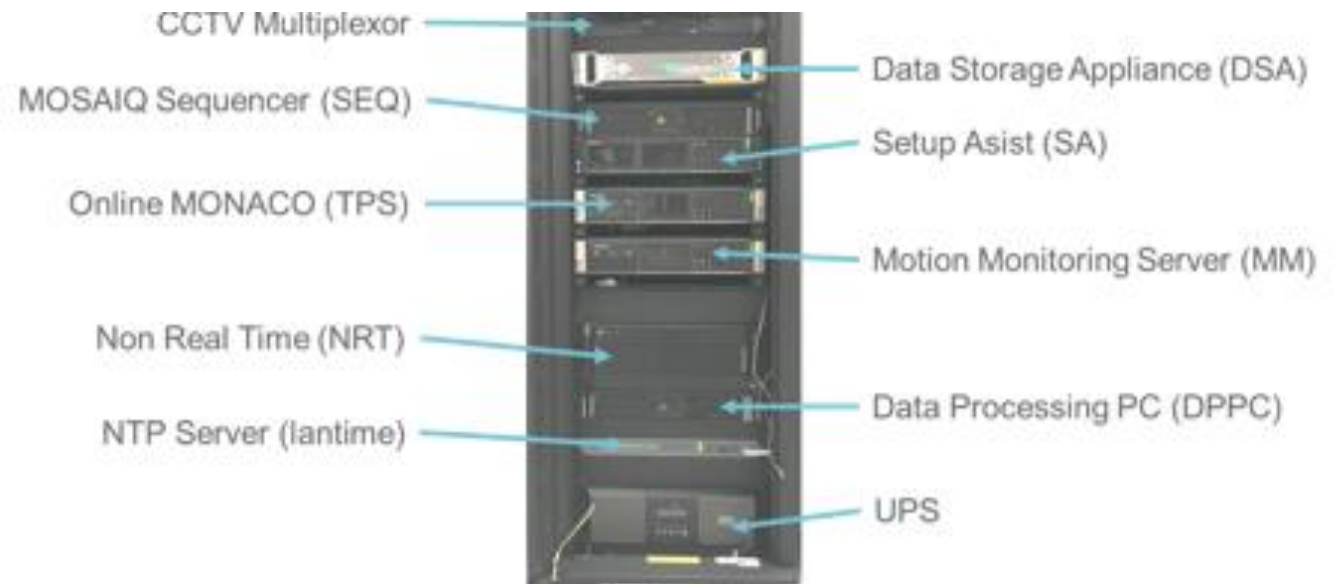
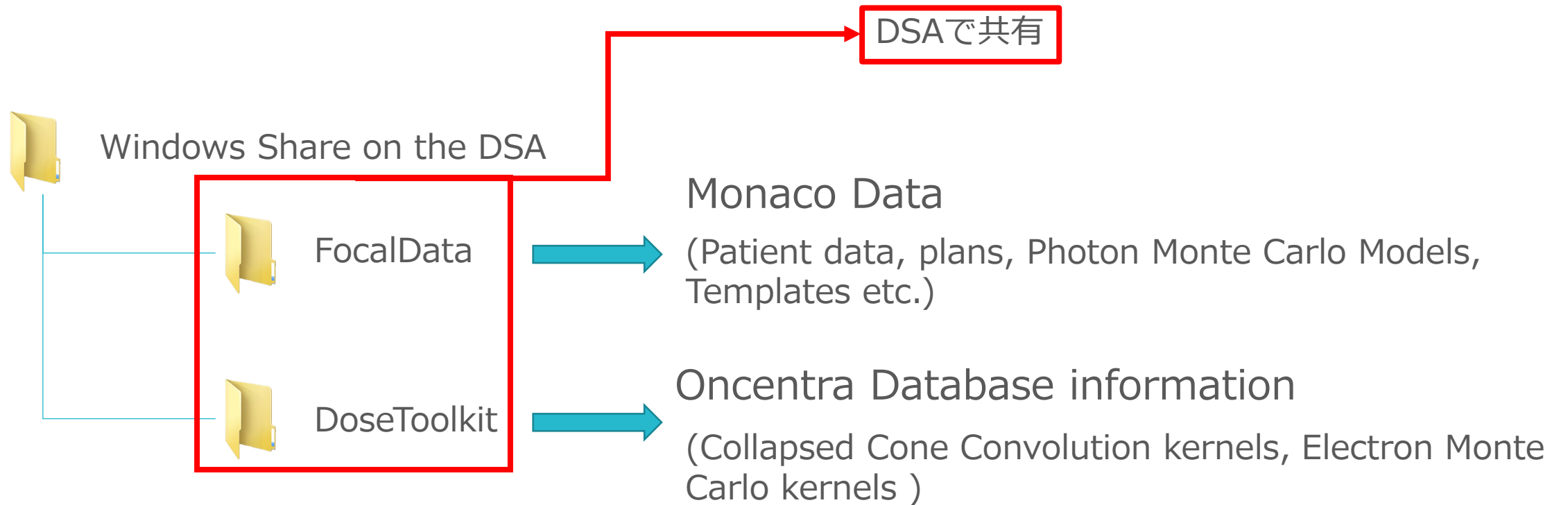
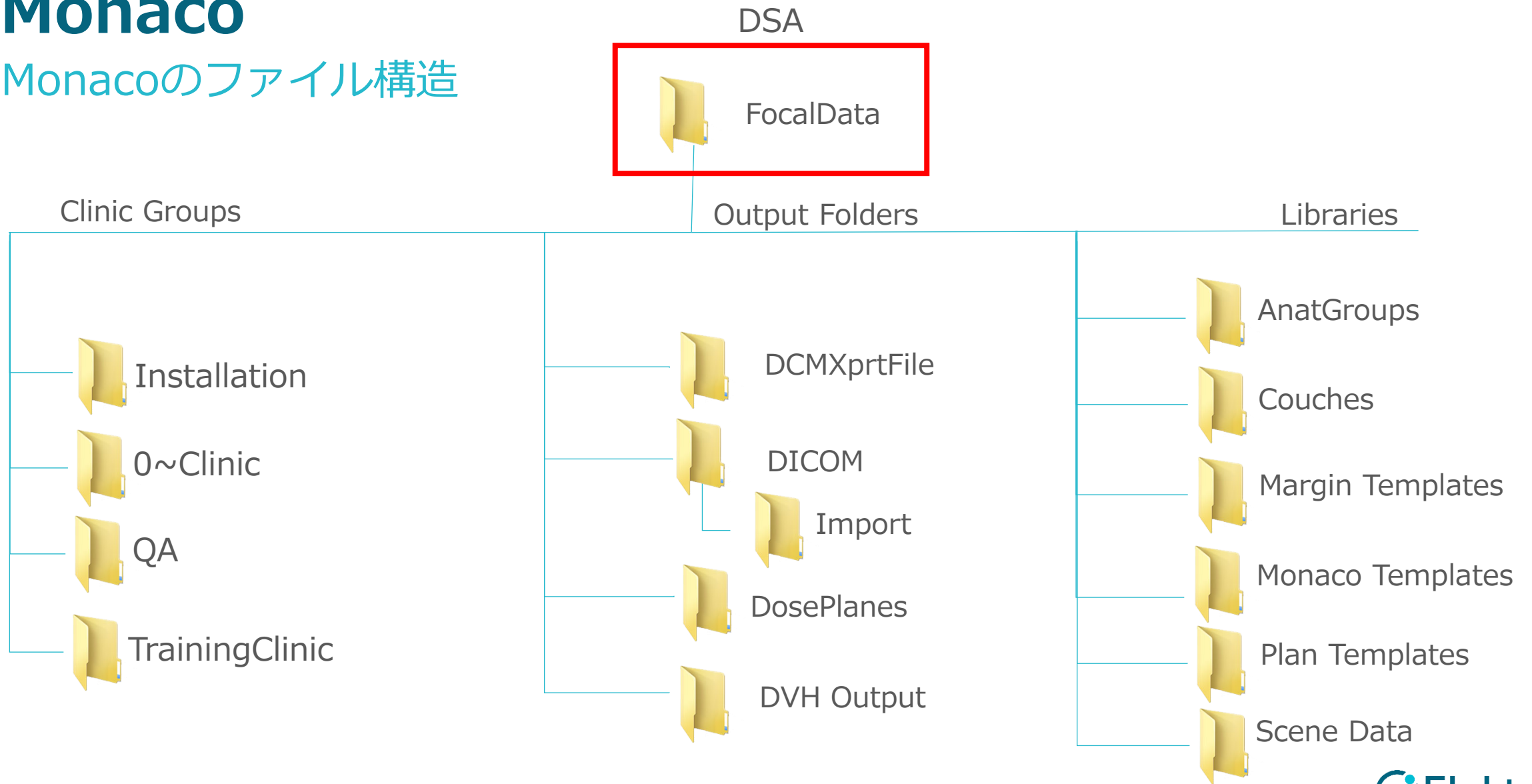
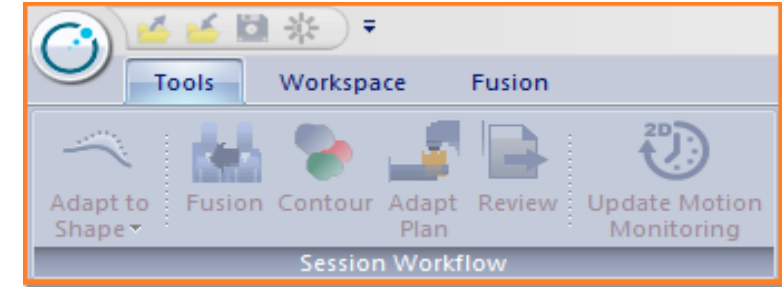
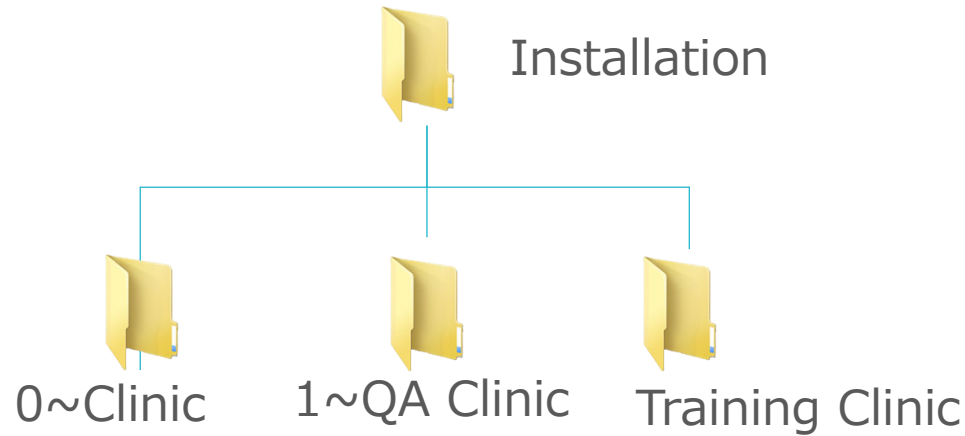
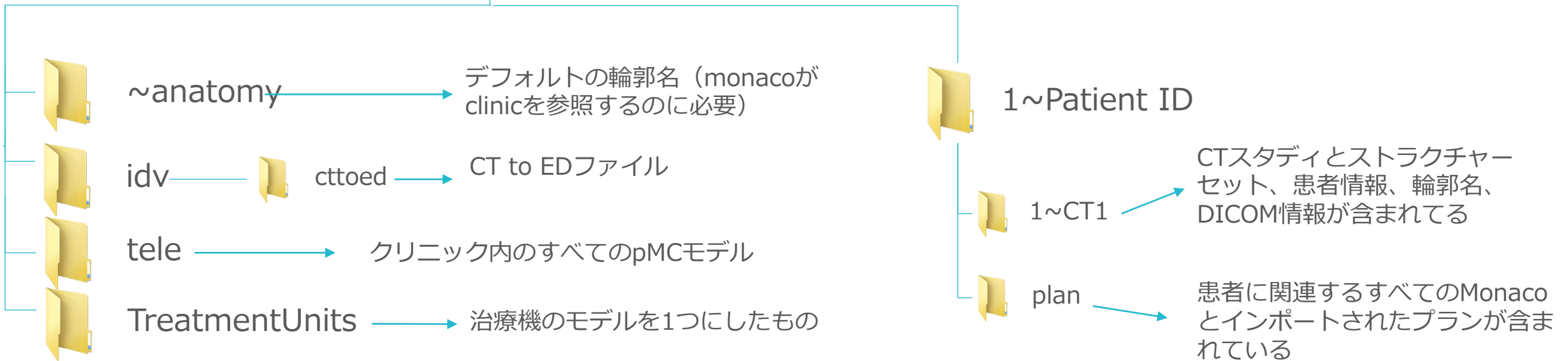
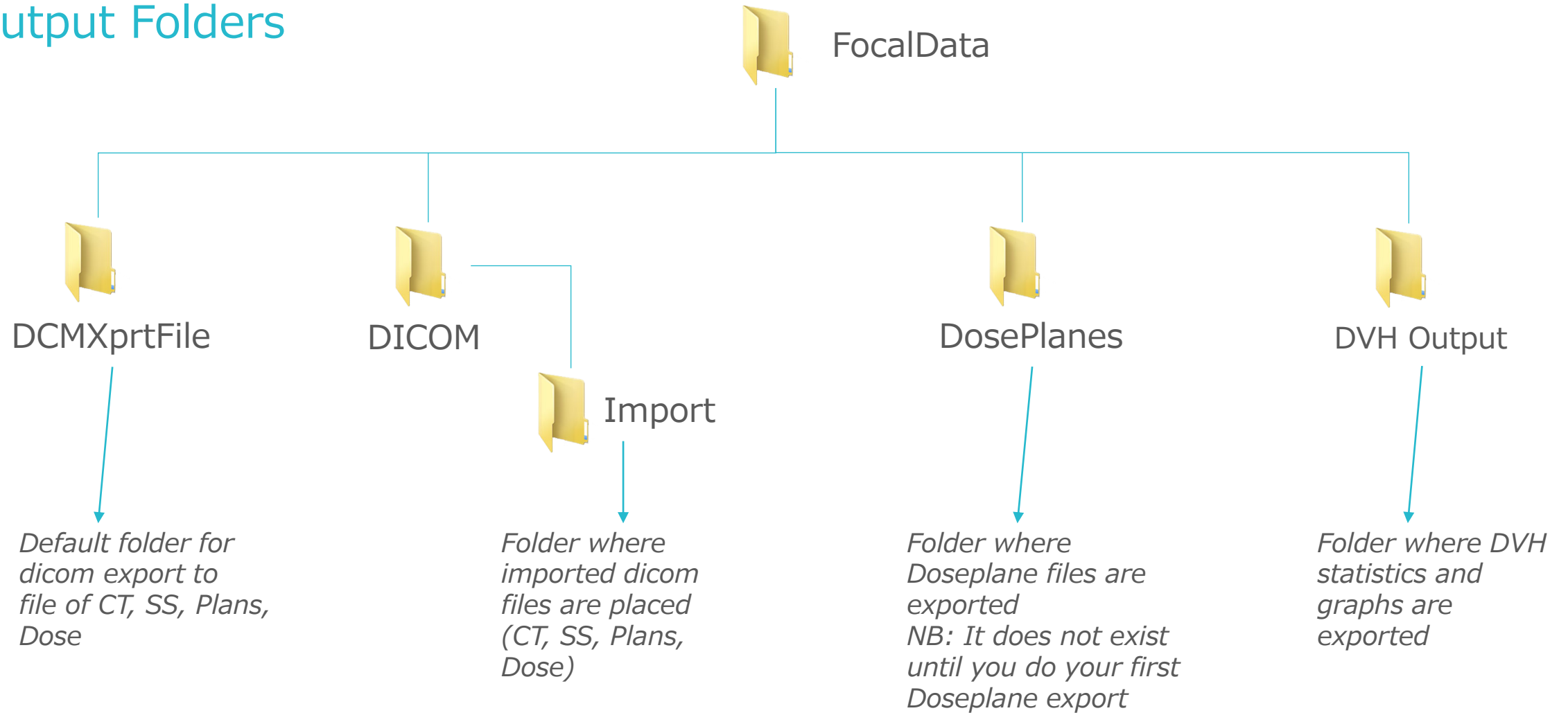
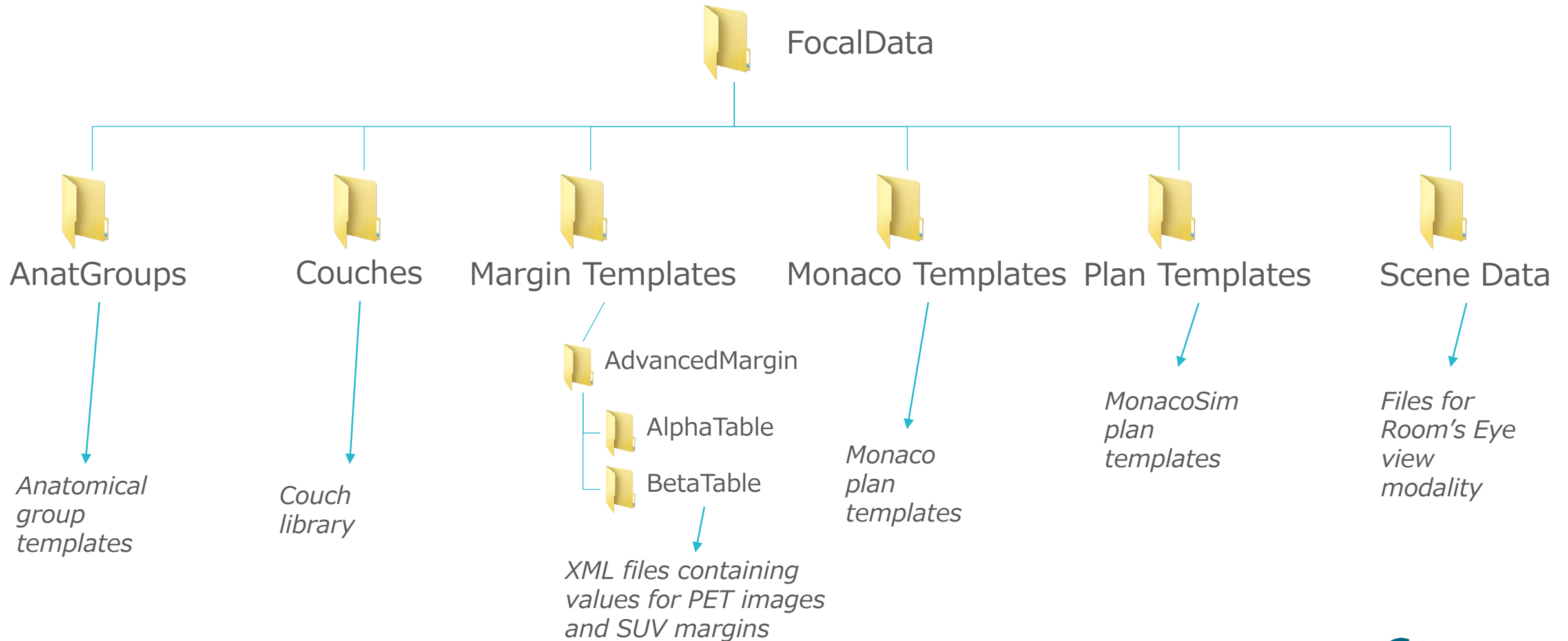
Libraries

Topics Covered

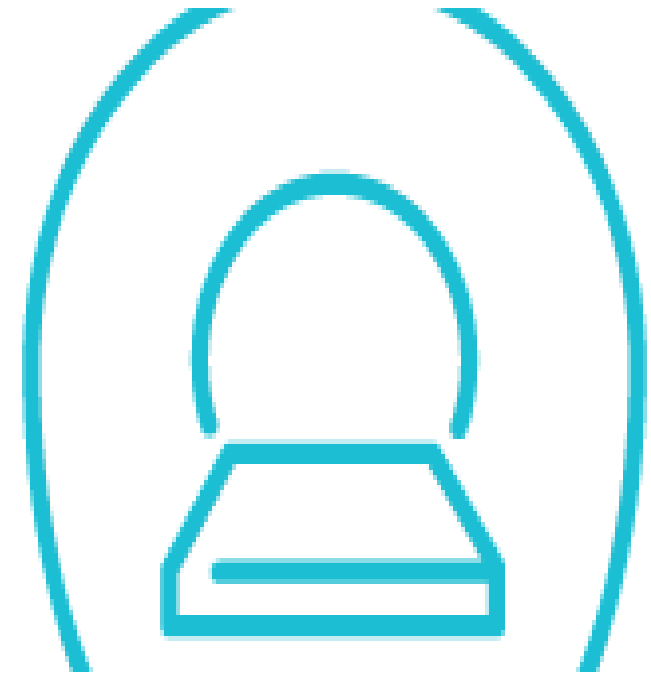- The relationship between online and offline Monaco
- Monaco Settings and User Authorization

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

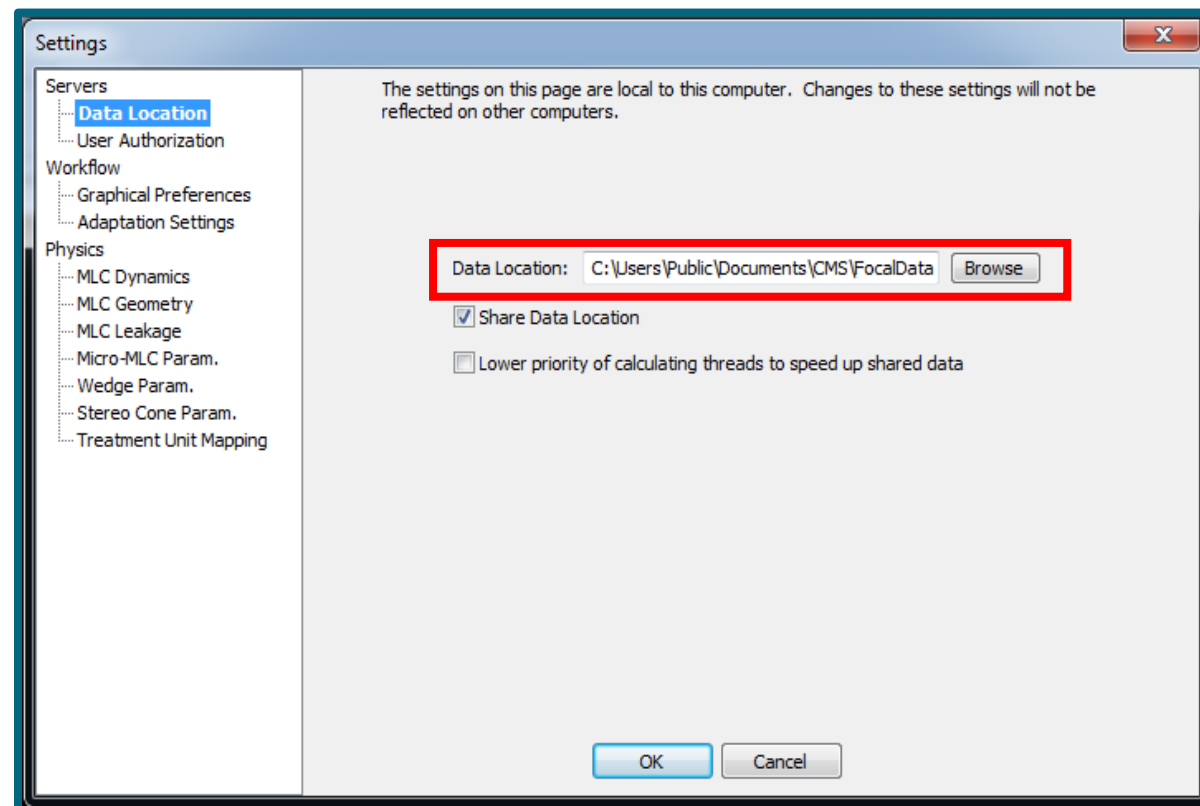
Monaco Configuration

Setup

- The Setup options for:
 - User Authorization
 - DICOM Settings
 - CT to ED Files

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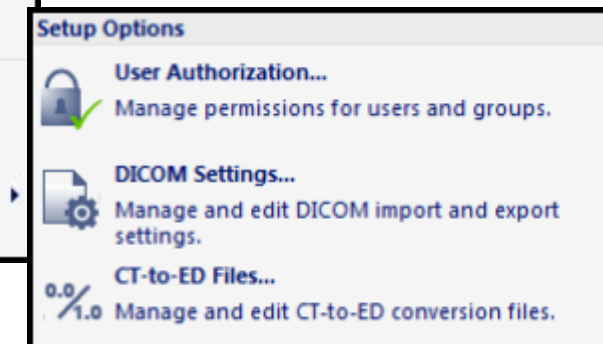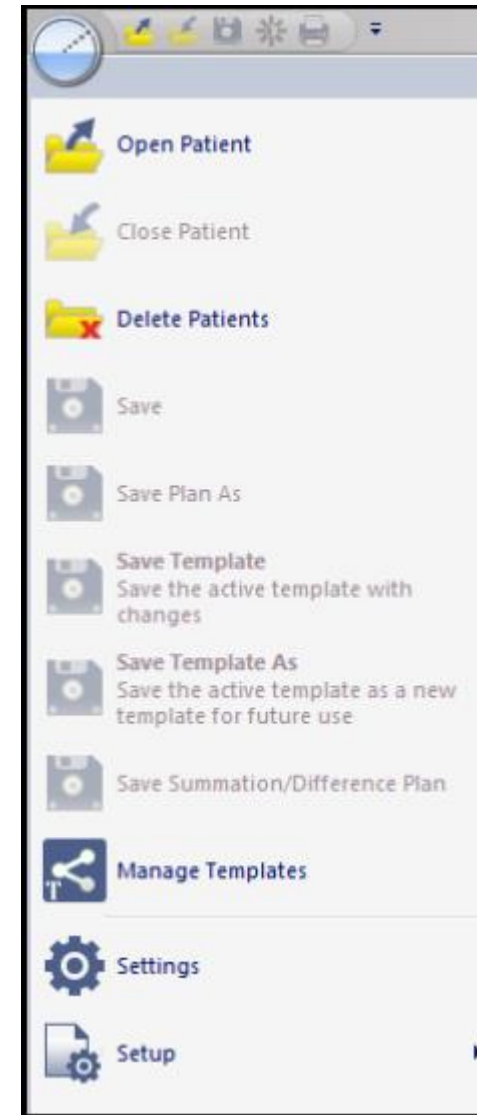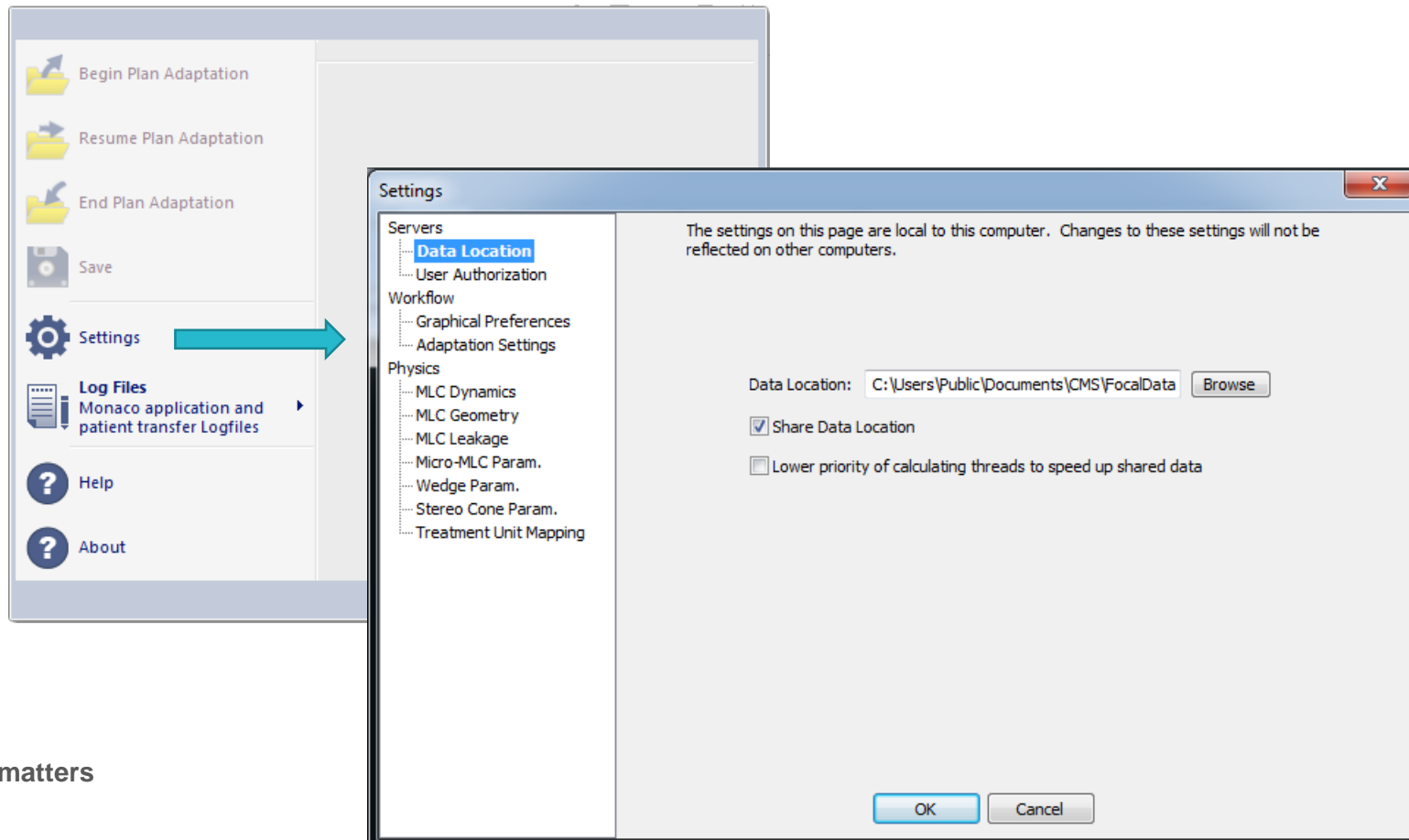
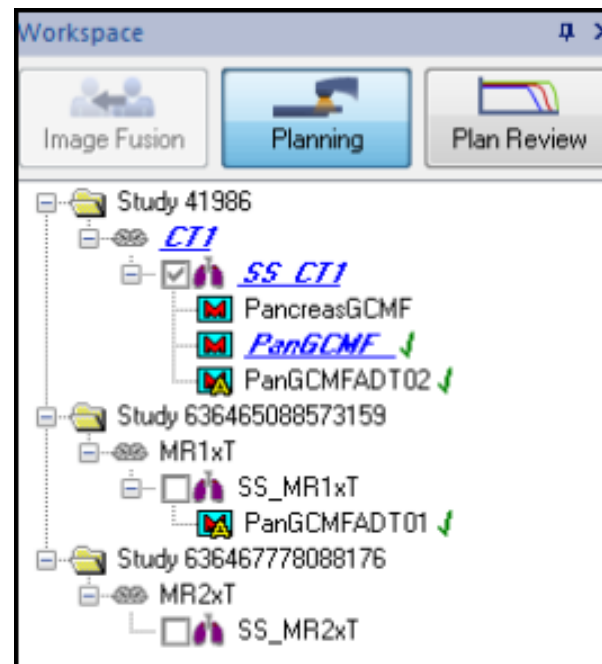
ACCT#:

DICOM Message Override Values:

Use Monaco Patient ID and Patient Name.

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- Monacoのsettingsで,Rx siteを追加可能

The image displays three screenshots from the Monaco software interface, illustrating the configuration of an Rx Site. A red circle highlights the 'Pancreas' site in the 'Additional Plan Export Options' window, with a red arrow pointing to the 'Pancreas' dropdown in the 'Radiation Prescription Sequencing' window.

Additional Plan Export Options

Course ID: 0
Plan Intent: Curative

Rx ID	Rx Site	Rx Dose (cGy)	Number of Fr...	Fractiona...	Dose Rate (M...	Tolerance Table
A	Pancreas	7400.0	37	200.0	0	Unity

Installation: Installation
Clinic: 0~Clinic

Rx Site Names:
Pancreas
Prostate

Radiation Prescription Sequencing - MR#: IDCIRS1 One, CIRS

Treat this Rx Site

Concurrent with beginning of
 Overlapping with
 After

Site: Pancreas

Treat the first fraction of this Rx Site at least
1 Day(s) After Fraction 37 of Site Pancreas

Sequencing will be saved only if you OK the Radiation Rx form

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- MonacoのsettingsでTolerance Tableを追加可能

The image shows two overlapping windows from the Monaco software. The top window is titled "Tolerance" and has the following fields:

- Department: Global - Available to All Departments
- Tolerance Type: Treatment Field, Conventional Modality
- Name list: Clinical (highlighted), Electron, IMRT, Photon, Unity
- Buttons: Close, Add, Change

The bottom window is titled "Additional Plan Export Options" and has the following fields:

- Course ID: 0
- Plan Intent: Curative
- Table with columns: Rx ID, Rx Site, Rx Dose (cGy), Number of Fr..., Fractiona..., Dose Rate (M..., Tolerance Table
- Table row: A, Pancreas, 7400.0, 37, 200.0, 0, Unity
- Buttons: OK, Cancel

Red circles and arrows highlight the "Unity" entry in the "Name" list of the "Tolerance" window, the "Unity" entry in the "Tolerance Table Names" list of the "Additional Plan Export Options" window, and the "Unity" entry in the "Tolerance Table" column of the table in the "Additional Plan Export Options" window.

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The settings on this page are local to this computer. Changes to these settings will not be reflected on other computers.

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